
Sidekick® All-In-One Blood Glucose Testing System

Test Strips Instructions for Use

Intended Use: Sidekick® All-In-One Blood Glucose Testing System quantitatively measures glucose in whole blood. Sidekick® All-In-One Blood Glucose Testing System is used for self-testing at home and professional use. Sidekick® All-In-One Blood Glucose Testing System is for *in vitro* (external) use only.

Test Principle: Sidekick® All-In-One Blood Glucose Testing System Test Strip is a plastic strip containing chemicals and electrodes. Sidekick® All-In-One Blood Glucose Testing System measures glucose by using amperometric technology employing a glucose oxidase reaction. When whole blood or Control is drawn into the tip of a Test Strip, glucose in the sample reacts with chemicals and produces an electrical current. The Meter measures electrical current and calculates amount of glucose. The result is displayed as a plasma value.

Chemical Composition: Glucose Oxidase (*Aspergillus sp.*) 2.5 Units, Mediators, Buffers, & Stabilizers.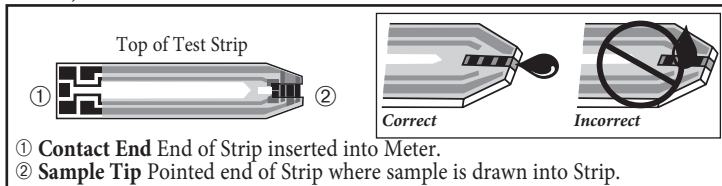

- ① **Contact End** End of Strip inserted into Meter.
 ② **Sample Tip** Pointed end of Strip where sample is drawn into Strip.

WARNING!

Upon opening the system carton, examine the product for missing, damaged or broken parts. Ensure the test strip vial cap is securely closed. If the product is damaged or the vial cap is not closed, DO NOT use the test strips for testing; product may give inaccurate results. Call 1-800-803-6025 for replacement and assistance.

Caring for Test Strips

- Use Strip quickly after removing from vial. Recap vial right away. Strips left outside of vial too long may give an error message.
- Test Strips **must** be kept in original, labeled vial, with vial tightly sealed. After removing Test Strip from vial, immediately replace cap.
- Never transfer Strips from one vial to another. Only store Test Strips in their original vial.
- Write date opened on vial when removing first Strip. Discard entire system 120 days after date opened.
- Store in a dry place at room temperature below 86°F/30°C. **DO NOT REFRIGERATE OR FREEZE.**
- DO NOT store in bathroom or kitchen.
- DO NOT expose to extreme heat or cold, direct sunlight, or high humidity for any period of time.
- DO NOT reuse Test Strips.
- DO NOT bend, cut or alter Test Strips in any way.

Important Information:

- Sidekick® All-In-One Blood Glucose Testing System Test Strips are for *in vitro* testing only. **DO NOT consume.**
- Sidekick® All-In-One Blood Glucose Testing System Test Strips in vial only work with Meter mounted on top of vial. Using other Test Strips or Control with Sidekick® All-In-One Blood Glucose Testing System may cause inaccurate results.
- Sidekick® All-In-One Blood Glucose Testing System is to be used **ONLY** with Sidekick® All-In-One Blood Glucose Testing System Test Strips and TRUEcontrol® Control Solution.
- NEVER** reuse Test Strips. **NEVER** wipe Test Strips with water, alcohol or any cleaner. **DO NOT** attempt to remove blood or control sample from Test Strips or clean Test Strips and re-use. Reuse of Test Strips will cause inaccurate results.
- NEVER** add a second drop of sample to Strip. Adding more sample gives an error message.
- Do not change your medication or treatment plan based on Sidekick® All-In-One Blood Glucose Testing System results without advice of Doctor or Diabetes Healthcare Professional.
- Discard used Test Strips and Lancets into an appropriate container (biohazard/for sharps objects).
- Using after expiration date printed on label or 120 days past opened date written on vial may cause inaccurate results.
- Discard any system that appears damaged.
- NEVER** use serum, plasma, or clotted blood when testing with Sidekick® All-In-One Blood Glucose Testing System. **ONLY** use fresh capillary whole blood from the finger or forearm.
- The Sidekick® All-In-One Blood Glucose Testing System is not recommended for use with venous samples.**
- Lancing device is for self testing and intended for use by one patient only. Not suitable for use by healthcare or care workers
- When using the forearm for blood sample:
 - Check with your Doctor or Diabetes Healthcare Professional to see if forearm testing is right for you.
 - Results from forearm are not always the same as results from fingertip.
- Use finger instead of forearm for more accurate results:
 - Within 2 hours of eating, exercise, or taking insulin,
 - If your blood sugar may be rising or falling rapidly or your routine results are often fluctuating,
 - If you are ill or under stress,
 - If your forearm test results do not match how you feel,
 - If your blood Sugar may be low or high,
 - If you do not notice symptoms when blood sugar is low or high.

Quality Control (QC) Testing:

There are two quality control tests to let you know that the System is working properly.

Quality Control Test: Automatic Self-Test


An automatic self-test is performed each time a Sidekick® All-In-One Blood Glucose Testing System Test Strip is inserted into the Meter. Upon inserting a Strip into the Test Port, if all segments appear and the Drop Symbol appears in the Display, the Meter is working properly.

Quality Control Test: Control

TRUEcontrol® Control Solution is used to check testing technique and System Performance. When Control results fall within ranges found on the Test Strip vial label of Test Strips being used, the System is working properly and testing technique is good. See System Instructions for Use or Control Solution Instructions for Use for more information on Control Testing.

Blood Glucose Testing

- Wash area to be lanced. Dry thoroughly.
 - Allow Sidekick® All-In-One Blood Glucose Testing System and Test Strips to sit at room temperature for 30 minutes before using. If opening Sidekick® All-In-One Blood Glucose Testing System for the first time, write date opened on vial label.
 - Check Expiration Date on Sidekick® All-In-One Blood Glucose Testing System being used.
Do not use if expired or 120 days past date opened.
 - Remove one Test Strip from vial. Recap vial immediately.
 - Insert Contact End of Strip, contacts facing into Test Port of Meter. Meter turns on. Do not remove Strip from Meter until testing is finished.
 - Obtain blood drop.
 - After Drop Symbol appears in Display, place Test Strip Sample Tip of strip gently against drop and allow blood to be drawn into Strip. Remove Test Strip Sample Tip from sample drop immediately after Meter starts to countdown on the Meter Display.
- CAUTION!** Holding the Test Strip Sample Tip to the blood sample too long after the Meter begins testing may cause inaccurate results.
If Meter does not begin testing 5 seconds after touching blood drop to Test Strip, see **Troubleshooting** in the Sidekick® All-In-One Blood Glucose Testing Instruction For Use.
- Result is displayed. Record Result.
 - Remove Strip from Meter. Discard used Test Strip and lancet in appropriate container. Meter turns off.

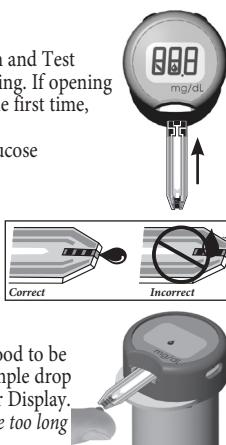Expected Results

Expected Results for people without diabetes:²

Plasma Blood Glucose Result	Plasma Blood Glucose Result
Before eating	< 110 mg/dL
Two hours after meals	< 140 mg/dL

A Doctor or Diabetes Healthcare Professional determines personal target glucose ranges. Always check with your Doctor or Diabetes Healthcare Professional before changing your treatment program.

If you are having symptoms that your glucose is too low or too high, contact your Doctor or Diabetes Healthcare Professional immediately.

The Sidekick® All-In-One Blood Glucose Testing System is optimized for testing with fresh capillary whole blood and calibrated to display a plasma result. Plasma results are estimated to be 10%-15% higher than whole blood results.³

If comparing results using Sidekick® All-In-One Blood Glucose Testing System results to laboratory results, perform a fingerstick blood test within 30 minutes of the laboratory test. Diabetes experts have suggested that glucose meters should agree within 15 mg/dL of a laboratory method when the glucose concentration is less than 75 mg/dL, and within 20% of a laboratory method when the glucose concentration is 75 mg/dL or higher.⁴ If you have eaten recently, results using Sidekick® All-In-One Blood Glucose Testing System can be up to 70 mg/dL higher than laboratory results.⁵

If glucose tests are performed using capillary blood with the Sidekick® All-In-One Blood Glucose Testing System and venous blood with the laboratory system, capillary blood (finger) may be up to 70 mg/dL higher than venous blood (vein), even when each test is performed within 10 minutes after sampling and per manufacturers' instructions.⁶

Troubleshooting (See System Instructions for Use for more details.)

If your result is unusually high or low or doesn't match the way you feel, perform a Control Test (see *Quality Control Testing*).

If the Control Test is within range:

- Read the testing procedure again.
- Test your blood glucose again using a new Sidekick® All-In-One Blood Glucose Testing System Test Strip.

If the results still do not match the way you feel, check with your Doctor or Diabetes Healthcare professional before changing your treatment program.

If the results are not within range:

- Check the Expiration Dates. Do not use if past either written date or date printed on the Test Strip vial or Control bottle label. Test with new Strips/Control.
- Check for error messages. If an error message appears, follow the actions in the *Display Messages* section of the Sidekick® All-In-One Blood Glucose Testing Instructions for Use.
- Check your testing technique. Perform another Control Test.
- Check the temperature. Allow System to reach room temperature 68°F- 77°F (20°C-25°C) before testing.

See Sidekick® All-In-One Blood Glucose Testing Instructions for Use for additional assistance with troubleshooting.

Limitations

Sidekick® All-In-One Blood Glucose Testing System products cannot be used for testing blood glucose in newborns.

The following **WILL** affect accurate test results:⁶

- Abnormally high doses of acetaminophen.
- Oxygen Therapy.

NOTE: Similar observations have been reported by other blood glucose systems.

The following will **NOT** affect accurate test results:⁶

- Hematocrit levels between 30% and 55%.
- Elevated total blood cholesterol and triglycerides.
- Salicylate occurring in expected blood concentrations.
- Testing at altitudes up to and including 10,150 feet.

DO NOT perform capillary blood glucose testing on critically ill patients.

Capillary blood glucose levels in critically ill patients with reduced peripheral blood flow may not reflect the true physiological state. Reduced peripheral blood flow may result from the following conditions (for example):⁷

- shock
- severe hypotension
- severe dehydration
- hyperglycemia with hyperosmolarity, with or without ketosis.

FOR CONSUMERS • Performance Characteristics

Accuracy

Accuracy describes how well Sidekick® All-In-One Blood Glucose Testing System results agree with results from an accepted laboratory analyzer. The accuracy of Sidekick® All-In-One Blood Glucose Testing System was assessed against Yellow Springs Instrument Model 23A, (which also uses a glucose oxidase method). Testing was performed by lay users at 4 independent clinical sites. The study demonstrates that Sidekick® All-In-One Blood Glucose Testing System compares well with the laboratory method. A regression analysis of the 229 results is given below:

N	Slope	y-intercept	r ²
229	0.98	5.18	0.96

Precision

Precision describes the variation between results. Precision results were performed in a laboratory.

Within Run Precision (whole blood)

	51 mg/dL	82 mg/dL	122 mg/dL	241 mg/dL	500 mg/dL
C.V. (%)	5.1	4.2	3.7	2.7	3.0

Total Precision (Control Solution)

	90 mg/dL	161 mg/dL	286 mg/dL
C.V. (%)	6.3	5.6	4.5

Consumer Study

Sidekick® All-In-One Blood Glucose Testing System was tested at diabetes clinics of a large urban area hospital and diabetes care centers. Sidekick® All-In-One Blood Glucose Testing System results were obtained from 212 patients and were compared to parallel results obtained on Yellow Springs Model 2300 STAT. This study shows that patients, when self-testing, and laboratory professionals are able to obtain accurate results when compared to results obtained from a laboratory system.

A clinical study was performed at an urban area hospital to compare forearm and finger stick blood glucose results using the Sidekick® All-In-One Blood Glucose Testing System. Sidekick® All-In-One Blood Glucose Testing System blood glucose tests were performed by patients self-testing (n=100) and on patients' blood by healthcare professionals (n=108). Results demonstrate substantial equivalence between forearm and finger blood glucose values when patients' blood glucose is in steady state and blood glucose tests are performed by patients or healthcare professionals using the Sidekick® All-In-One Blood Glucose Testing System.

See your Sidekick® All-In-One Blood Glucose Testing System Instructions for Use for detailed instructions. Call 1-800-803-6025 for assistance, Monday - Friday, 8AM-8PM EST.

For medical assistance, contact a Doctor or Diabetes Healthcare Professional.

References

- U.S. Food and Drug Administration. Blood Glucose Meters, Getting the Most Out of Your Meter. [Electronic Version]. Retrieved December 22, 2009 from <http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/TipsandArticlesonDeviceSafety/ucm109371.htm>.
- Joslin Diabetes Center. Goals for Blood Glucose Control [Electronic Version]. Retrieved July 25, 2011 from <http://www.joslin.org/info/Goals-for-Blood-Glucose-Control.html>.
- Sacks, D.B., "Carbohydrates," Burtis, C.A., and Ashwood, E.R. (ed.), Tietz Textbook of Clinical Chemistry. Philadelphia: W.B. Saunders Company (1994), 959.
- International Organization for Standardization. In vitro diagnostic test systems. Requirements for blood-glucose monitoring system for self-testing in managing diabetes mellitus. Reference number ISO 15197:2003 (E). Geneva: International Organization for Standardization; 2003.
- Larson-Cohn U: Difference between capillary and venous blood glucose during oral glucose tolerance tests. *Scand J Clin Lab Invest* 36:805-808, 1976.
- Data on file.
- Atkins, S. H., Dasmahapatra, A., Jaker, M.A., Chorost, M. I., Redd, S. Fingerstick Glucose Determination in Shock. *Annals of Internal Medicine*, 114:1020-1024, 1991.

Manufactured by:
 Trividia Health, Inc.
 Fort Lauderdale, FL 33309

© 07/2018 Trividia Health, Inc. Sidekick is a trademark of Trividia Health, Inc.

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Referencias

- U.S. Food and Drug Administration. Blood Glucose Meters, Getting the Most Out of Your Meter. [Versión electrónica]. Obtenido el 22 de diciembre de 2009 de <http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/TipsandArticlesonDeviceSafety/ucm109371.htm>.
- Joslin Diabetes Center. Goals for Blood Glucose Control [Electronic Version]. Obtenido el 25 de julio de 2011 de <http://www.joslin.org/info/Goals-for-Blood-Glucose-Control.html>.
- Sacks, D.B., "Carbohydrates," Burtis, C.A., and Ashwood, E.R. (ed.), Tietz Textbook of Clinical Chemistry. Filadelfia: W.B. Saunders Company (1994), 959.
- International Organization for Standardization. In vitro diagnostic test systems. Requirements for blood-glucose monitoring system for self-testing in managing diabetes mellitus. Reference number ISO 15197:2003 (E). Ginebra: International Organization for Standardization; 2003.
- Larson-Cohn U: Differences between capillary and venous blood glucose during oral glucose tolerance tests. Scand J Clin Lab Invest 36:805-808, 1976.
- Información archivada.
- Atkins, S. H., Dasmahapatra, A., Jaker, M.A., Chorost, M. I., Redd, S., Fingertstick Glucose Determination in Shock. Annals of Internal Medicine, 114:1020-1024, 1991.

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